

VaultStream® EasyCut™

Medical Video Editing Application for Clinicians

FEATURES:

- Supports industry-standard MPEG-4 High Definition (HD) videos.
- Rough & fine-cut editing for MPEG-4/H.264 clinical videos.
- Preserves HD quality with loss-less editing.
- Compatible with Mac® and Windows®
- Microsoft® PowerPoint® integration.
- One-click automatic edit mode.
- Synchronized, lock-step editing.
- Multi-Movie Option.
- Integrated with VaultStream.


EasyCut is a powerful medical video editing and production software application that helps clinicians create their own presentation-ready videos.

Coupled with Image Stream Medical’s video management library (VaultStream®) and recording platforms, EasyCut yields time-savings to busy clinicians who rely on high-quality video presentations for teaching, professional meetings, surgical documentation, and collaborations with colleagues.

Connect to the VaultStream server to access saved video.

Edit multiple movies simultaneously.

Create clips from the source video.

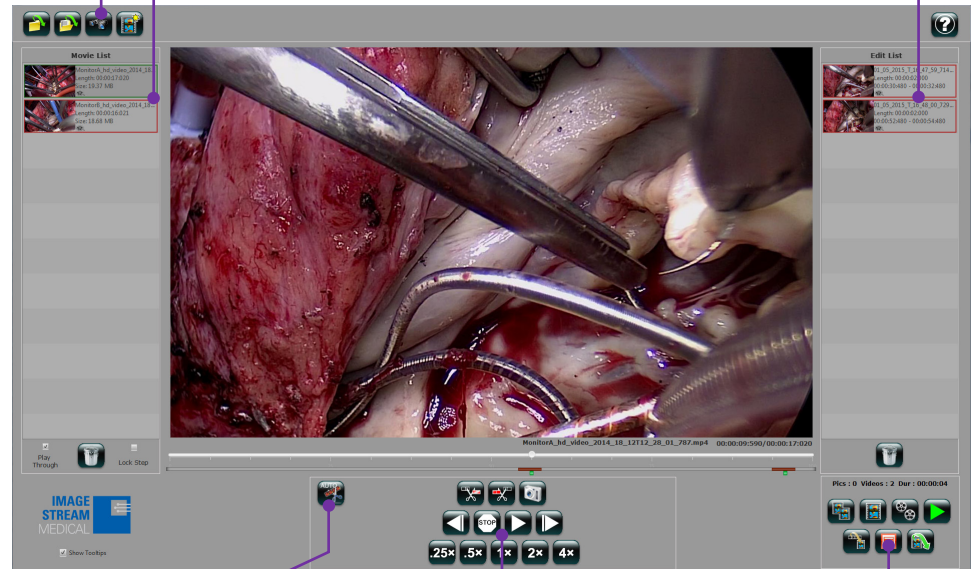

Automatic video editing.

Precision navigation to get just the clips needed.

Export automatically to PowerPoint.

Save Time And Improve Security

Most video editors require a local copy of the video. This means users need a media source, often an unsecured USB drive, to copy their video to a local PC. EasyCut requires no additional media. Instead, users edit videos directly from VaultStream, optionally copying locally only the clips they want to save. Avoiding a lengthy download, lets users begin editing immediately.

Point of Care Editing with AutoEdit

With Image Stream Medical’s recording platforms, clinicians can bookmark notable findings during the procedure. EasyCut visually highlights bookmarks on the procedure timeline so clinicians don’t have to guess when they occurred. The EasyCut AutoEdit button automatically creates video clips at each bookmark, enabling clinicians to quickly save and share the most clinically important portions of each procedure.

High Speed Navigation With Precise Control

EasyCut helps users quickly find the content they need with ¼ (0.25x) to quad (4x) speed and frame-by-frame navigation options. On-screen video slider and Cut-In and Cut-Out buttons enable precise clip location and length selection.

Podium-Ready Presentations In Minutes

Users can export clips and images directly to PowerPoint to create a podium-ready presentation, optionally creating a template with their organization's logo for a customized look. Drag and drop videos and images into applications such as Apple® Keynote® and iMovie®.

Lock-Step Editing For Multi-Source Recordings

For procedures with multiple video sources, it's important to synchronize each recording at the same point in time. EasyCut's lock-step mode lets clinicians edit the primary source video and then duplicates those edits across the other video streams with a single click, eliminating the error-prone task of editing multiple streams.

Preserve High-Definition Video Quality

Some video editors decompress and recompress video which can degrade image quality. EasyCut edits video in its native, industry-standard MPEG-4 format, maintaining High-Definition video quality no matter how many edits are required.

Additional Features:

- Edit multiple procedures in same project.
- Save clips as separate movies or stitch-together into a single movie.
- Import both video and snapshot images from VaultStream.
- Capture and save images while editing.
- Submit new content to non-clinical, searchable VaultStream teaching libraries.
- Play-through option sequentially plays through multiple clips without pause.
- On-screen bookmark and clip location highlights.
- Multi-movie feature automatically creates a separate movie, containing all the clips extracted from a single source when editing multiple movies at once.
- Edit from local disk or network VaultStream.
- Easily install to multiple Windows systems using Microsoft Group Policy.

COMPATIBLE WITH:

- VaultStream V8.0 Video Library, VaultStream V7.0 Video Library (Some features require Vault Stream V8.0 Video Library).
- Projects created with EasyCut V6.0.
- EasySuite RX, nCare MX and nStream GX Recording & Capture Platforms.

REQUIREMENTS

Software

Operating System

- Windows 7 & 8
- Mac OS-X® Maverick (10.9) & Yosemite (10.10)

Microsoft PowerPoint

- Required for PowerPoint export feature

Microsoft Office

- Windows 2010, 2011, 2013
- Mac OS 2011 2013

Adobe® Acrobat®

- Current version installed

Hardware

Graphics Adapter

- Windows DirectX® 11 Compatible with 128 MB+ Ram

Memory

- 32-bit OS 2 + GB RAM
- 64-bit OS 4+ GB RAM

Hard Drive

- 60 GB minimum (250 GB recommended)

Screen Resolution

- 1280 x 1024 (1920 x 1080 or 1920 x 1200 recommended)

Network

- 1Gb/s Ethernet



One Monarch Drive
Littleton, MA 01460
T: 978.486.8494
F: 978.428.2694

1002042 Rev. A

www.ImageStreamMedical.com
info@imagestreammedical.com